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Application and safety instructions for the MEISINGER

Master-Mill

Surgical Bone Mill

Art.-No.: BKM00

The Bone Management® Master-Mill is a surgical bone mill used for crushing autologous bone blocks and thus preserving small autologous bone chips. With the help of these bone chips, bone defects of all shapes can be optimally filled. The individual components of the Master-Mill are made of high quality, surgical stainless steel. All parts feature an optimal fit and a stable geometry, allowing the Master-Mill to be easy to handle with ease of assembly and intuitive use.

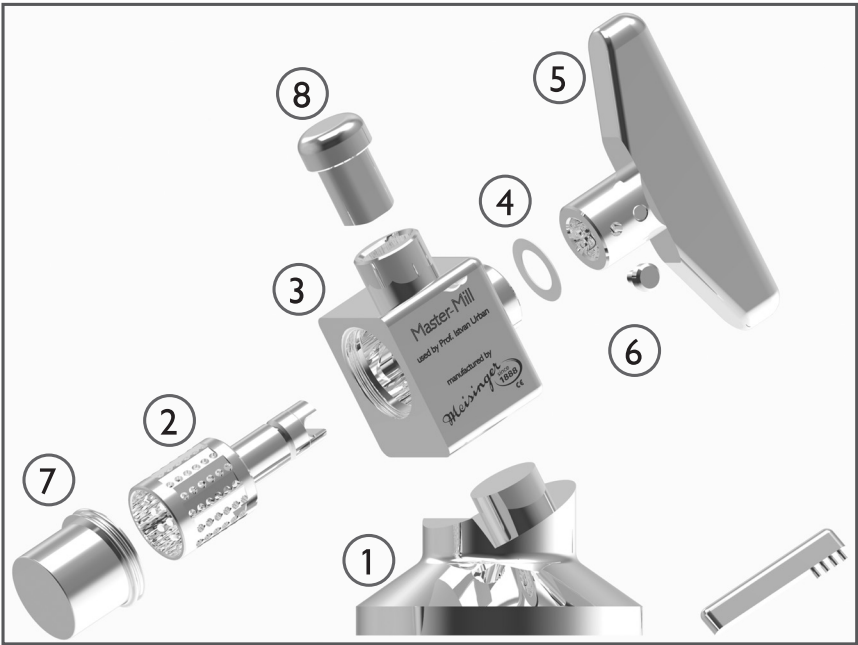
Indications for Use

The Bone Management® Master-Mill is indicated for crushing autologous, corticocancellous bone blocks into bone chips. The resulting bone chips are suitable for immediate use for defect filling in the context of bone augmentation.

Utilization

Preparation of the Master-Mill

The Master-Mill is delivered disassembled. The components are packed unsterile and must be processed before the first as well as before any further use, according to instructions of processing (cleaning, disinfection, sterilization).

- After processing, the individual components of the Master-Mill are assembled as follows:
- Place the pedestal (1) on a stable, even and horizontal surface.
 - Insert the grinder (2) with the extension first into the housing (3).
 - Put the sliding disc (4) onto the extension of the grinder (2) until it lies flatly against the housing (3).
 - Put the handle (5) onto the extension of the grinder (2). The handle must engage so that it lies flush against the housing (3). Then hand-tighten the fixing screw (6) on the handle (5) so that handle, grinder and housing are connected to each other.
 - On the opposite side of the handle, hand-tighten the collector cup (7) to the housing (3).
 - Insert the stamp (8) in the upper opening of the housing (3).
 - Finally, place the housing (3) on the pedestal (1) with the handle (5) and stamp (8) facing upwards.

Damage to the individual components can have a negative impact on the fit and thus on the function of the bone mill. Therefore, check the function of the Master-Mill by turning the handle before using it. It should be possible to turn the grinder easily in the housing.

Crushing of bone blocks

To use the Master-Mill, remove the stamp and place an extracted bone block in the opening on the grinder. Then close the opening with the stamp again. The grinder is turned using the handle. With the help of the stamp, pressure must be exerted on the bone block so that it is grasped and crushed by the toothing of the grinder. During use, the grinder should be turned both clockwise and counterclockwise to prevent jamming of the bone block.

Extraction of the crushed bone

The bone chips resulting from the grinding process are collected in the collector cup and are ready for further use. Remaining bone in the grinder can be removed from the grinder with the help of the bone residue remover, which also consists of surgical stainless surgical steel. To do this, the grinder must be removed from the housing in accordance with the instructions for assembly.

Disassembly of the Master-Mill after use

For processing the Master-Mill is to be completely disassembled after use, following the assembly instructions.

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