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Application and safety instructions for the MEISINGER

Alveoplasty Surgical Kit

Surgical Contouring and Modification Kit

Art.-No.: ALV18

The Alveoplasty Surgical Kit contains the most important instruments for surgical contouring and modification of the alveolar ridge when unwanted unevenness of the jawbone occurs after tooth extraction or bone degeneration. In this case, the kit offers the best way to reshape the jawbone. The Alveoplasty Kit contains two initial burs and two Lindemann burs, which allow for the performance of precise, initial osteotomies, even in hard cortical bone. To perform the alveoplasty itself, five different carbide cutters and a round diamond are included. The round working parts optimally protect the soft tissue against injuries. Thus, the alveolar ridge is prepared carefully and precisely for denture restoration.

Indications for Use

The Bone Management® system Alveoplasty Surgical Kit is indicated for surgical contouring and modification of the alveolar ridge after tooth extraction or in preparation for the restoration with a total denture.

Utilization


The recommended speeds for the application and the maximum speeds are summarized in the instrument overview under „Content“. For proper, safe use, these must be observed.

Performance of the alveoplasty


Reshaping of the alveolar ridge surface

Irregularities on the jawbone can be easily contoured by trimming the bone surface. After exposing the bone, the carbide instruments, as well as the diamond instrument are used to contour or shape the bone in the desired manner. Especially with the instruments HM72GX, HM77GX, HM251GX, HM71 and 801 the large-area treatment or contouring of the alveolar ridge is possible. The round working part shapes of the instruments protect against soft tissue damage. After completion of the treatment, the repositioning and the primary closure of the soft tissue are performed.


Alveoplasty as a stand-alone procedure: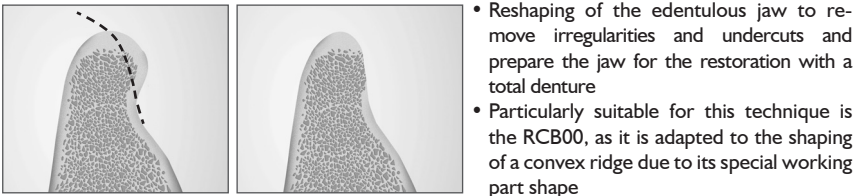

Alveoplasty after tooth extraction:

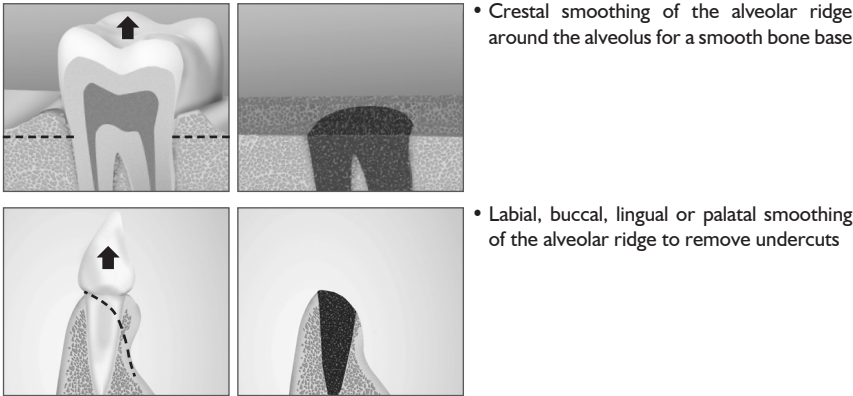Compression Technique

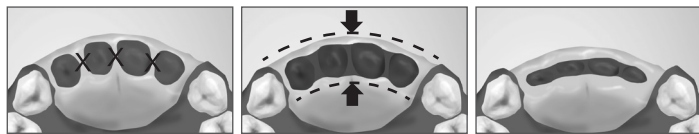The compression technique may be used when the labial, buccal, lingual or palatal plate of the alveolar ridge shows undercuts after tooth extraction. But instead of removing the cortical plate of the bone from the surface, the cortical plate is preserved with this compression technique. For this purpose, the protruding bone is first undermined (e.g. with the Lindemann cutters HM166 and 165RF). With the aid of the Initial Bur 186RF, the bone can be pre-drilled at the desired location. In addition, the bone occupying the space between adjacent empty dental alveoli is removed. Subsequently, the protruding portions of the ridge can be compressed by manual compression, so that undercuts are removed and the alveolar ridge receives a convex shape.

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Contraindications

Basically, general medical as well as local, absolute and relative contraindications for dental surgical procedures must be considered.

Absolute contraindications

- Dentoalveolar growth which has not come to an end (exception: cases in which no dentoalveolar growth can be expected, e.g. ectodermal dysplasia)
- Blood Dyscrasias / Blood Clotting Disorders
- Uncontrolled Diabetes
- Acute infections as well as pathological processes
- Insufficient bone volume (quality / quantity)

Relative contraindications

- Known allergies / foreign body reaction to used materials
- Diseases that affect bone metabolism
- Drug and alcohol abuse
- Lack of cooperation of the patient
- Poor circulation
- Hard physical labor or active sports
- Mental condition, which can lead to disregard of the medical order
- Highly atrophic jaw

IMPORTANT: Attention must be paid to protecting anatomical structures (safety clearance at least 2 mm) as well as the gradient of the adjacent tooth / roots (risk of damage, infection / dehiscences).

CAUTION: For all cutting and drilling procedures, the following applies: In order to reduce the risk of overheating the bone and necrosis formation along with it, the respective instrument has to be used with little contact pressure and continuous cooling using a sterile physiological saline solution. Furthermore, high contact pressure can also lead to the loss of smooth running, particularly when using the carbide cutters or the RCB00. All articles are to be used clockwise. For the carbide cutters and the RCB00, a pulling application (towards the user) is recommended.

CAS: 7440-48-4
Contains hazardous substances: Cobalt, CAS: 7440-48-4

This product contains more than 0.1 % by mass of cobalt and therefore must be labelled as CMR substance class 1B (carcinogenic, mutagenic, reprotoxic) according to current regulations. It has been demonstrated that there is no increased risk of cancer or adverse reproductive or genetic effects when used as directed.

ATTENTION: Any serious-incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

General instructions

Please follow general application and safety instructions for MEISINGER products in the medical area and also the advice for processing (cleaning, disinfection and sterilisation) of medical devices from Hager & Meisinger GmbH.

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