

Comprehensive Access Kit

Easy and safe endodontic procedures

Safe access into the root canal is imperative for any successful root canal treatment. That is why MEISINGER created the Comprehensive Access Kit, a carefully curated selection of diamond and tungsten carbide instruments necessary for a safe and predictable endodontic access preparation. This kit

provides all the burs needed to obtain straight line access into the canal through different materials such as zirconia, PFMs, and enamel. Conveniently organized in an autoclavable bur block, this kit is essential for any root canal treatment.



ENDO1









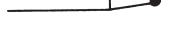


At a glance

- Obtain a straight line access to the root canal
- Remove the roof of the pulp chamber
- Cut through zirconia, PFMs, and enamel
- Expand and flare axial walls
- Uncover hidden orifices

Our sales force would be pleased to answer your questions: +49 2131 2012-303

Meisinger since 1888

Comprehensive Access Kit Fax for ordering +49 2131 2012-222

	Figure	Shank	Size	recomm. / max. revolutions	Pieces / Kit	Amount 
	ENDO1	-	-	-	-	
	HM1	FG XL (316)	010	1.000-1.500 min⁻¹ / 300.000 min ⁻¹	1	
	HM1	FG XL (316)	014	1.000-1.500 min⁻¹ / 300.000 min ⁻¹	1	
	Z801L	FG (314)	014	300.000 min⁻¹ / 450.000 min ⁻¹	1	
	856G	FG (314)	016	300.000 min⁻¹ / 450.000 min ⁻¹	1	
	HMG152	FG L (315)	016	1.000-1.500 min⁻¹ / 300.000 min ⁻¹	1	
	801G	FG (314)	010	300.000 min⁻¹ / 450.000 min ⁻¹	1	
	801G	FG (314)	014	300.000 min⁻¹ / 450.000 min ⁻¹	1	
	833G	FG (314)	023	300.000 min⁻¹ / 450.000 min ⁻¹	1	
	HM31A	FG (314)	012	160.000-230.000 min⁻¹ / 300.000 min ⁻¹	1	

CE 0044

Available from the dental trade.

Your trade depot:

Your delivery address:

Hager & Meisinger GmbH

Hansemannstr. 10 • 41468 Neuss • Germany
 Phone: +49 2131 2012-0 • Fax: +49 2131 2012-222
 info@meisinger.de • www.meisinger.de

Meisinger USA, L.L.C.

10200 E. Easter Avenue • Centennial, Colorado 80112 • USA
 Phone: +1 (303) 268-5400 • Fax: +1 (303) 268-5407
 Toll free: +1 (866) 634-7464
 info@meisingerusa.com • www.meisingerusa.com

85FL124EN - 0318