Inhalt Crest Control

Content Crest Control Contenido Crest Control

	Diamantierte Kugel Diamond-headed bur Bola diamantada	Osteotomieinstrumente Osteotomy instruments Fresas para osteotomia	Vorkörner Initial bur Fresa inicial		
Fig.	802L	HM33IL	859L	859	186RF
Shank ¹	205	205	204	204	204
Size ²	014	010	010	018	018
Length mm	8.0	5.5	12.0	10.0	12.0

¹ 204=RA, 205=RA L, ² Largest working part diameter in 1/10 mm

	Adapter Adaptors Adaptadores	Diamantierte Sägen Diamond saws Sierras diamantadas			
Fig.	SW0K1	SW0L1	231DC	231DC	231DC
Shank ¹			204	204	204
Size ²			070	100	130
Length mm			0.3	0.3	0.3

¹ 204=RA, 205=RA L, ² Largest working part diameter in 1/10 mm

Horizontal-Spreader
Horizontal spreaders
Spreader horizontal
(3 Stck. / 3 p. / 3 pzs.)

	Horizontal-Spreader Horizontal spreaders Spreader horizontal (3 Stck. / 3 p. / 3 pzs.)	Ratsche Ratchet Carraca
Fig.	VCD50	CARA4
Width	6.0	
Length	9.75	
Height	10.6	84.0

Inhalt Crest Control Basic

Content Crest Control Basic Contenido Crest Control Basic

	Chirurgiefräser aus Hartmetall Surgical Cutter of Tungsten Carbide Fresas quirúrgicas de carburo	Osteotomieinstrument Osteotomy instrument fresa para osteotomia	Diamantierte Säge Diamond coated saw Sierras diamantada	Adapter Adaptor Adaptadores
Fig.	HM 254E	859L	231DC	SW0K2
Shank ¹	205	204	204	
Size ²	012	010	100	
Length mm	6.0	12.0	0.3	

¹ 204=RA, 205=RA L, ² Largest working part diameter in 1/10 mm

Horizontal-Spreader
Horizontal spreaders
Spreader horizontal
(2 Stck. / 2 p. / 2 pzs.)

	Horizontal-Spreader Horizontal spreaders Spreader horizontal (2 Stck. / 2 p. / 2 pzs.)
Fig.	VCD50
Width mm	6.0
Length mm	9.75
Height mm	10.6

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- Controlled spreading of horizontally resorbed bone
- Procedures that are systematic and easy to learn
- Effective in increasing the width of narrow alveolar ridges
- Condensing the bone results in greater primary stability
- Preparation of the bone for all standard forms of implant systems
- Minimally invasive treatment concept
- Gentle spreading of the alveolar ridge.

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Horizontal Bone Spreading Systems

85FLBM02 - 0315

Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

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Crest-Control is a system for the controlled, pre-determinable widening of horizontally resorbed bone and is used in the distal mandible over a longer distance. The reliable horizontal spreaders make spreading of the alveolar ridge possible by up to 5 mm. Subsequently, an implant of all current systems can still be used during the mineralisation phase in the widened segment from which the osteointegration due to a highly-potent body growth factor of the existing callus may be accelerated.

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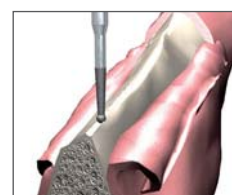
Indications of the use of Crest-Control systems include horizontally resorbed sections, e.g. of the mandibular alveolar ridge that do not allow for a conventional implantation to be performed without appropriate Bone Management®. Furthermore, Crest-Control is particularly suitable when inserting several adjacent implants.

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The first cutting line is made on the alveolar ridge with a diamond-coated ball in order to prevent slippage of the diamond-coated saw that is subsequently used.

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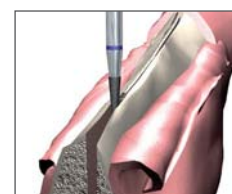
Along the cutting line, a crestal relief cut is made using the diamond saw. This promotes dilation of the bone.

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
In the area of the implant opening, with the help

of the osteotomy instrument, crestal cut can be deepened and prepared accordingly for the application of the horizontal spreader.

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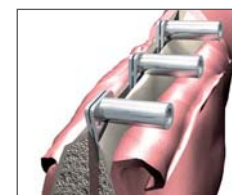
Following this the horizontal spreaders are positioned in the crestal cut.

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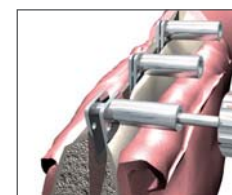Using the right screw driver, the horizontal spreaders are now spread very slowly, thus opening the gap. During this process, the bone is horizontally condensed at the same time.

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The now widened implant site enables the insertion of the suitable implant. In addition, the bone strength clearly due to the condensation offers an optimized primary stability.

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A further application possibility to the Horizontal Augmentation with the Crest-Control is found in the implementation according to the biological protocol of the Callus-Distraktion.

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- Insufficient supply of bone: Residual bone height < 10 mm / alveolar ridge width < 2 mm
- Implementation of the procedure directly after extraction or in the case of instable cortical bone
- Diseases that impair bone metabolism
- Dentoalveolar growth which has not come to an end (exception: cases in which no dentoalveolar growth can be expected, e.g. ectodermal dysplasia)
- Active infections as well as pathological processes

NOTE: Related contraindications and further instructions that are deemed set-specific and generally valid can be found at the download area of our website at "Application and Safety Instructions for Bone Management® Systems".

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