

Inhalt

Content Contenido

Elevatoren
Elevators
Elevadores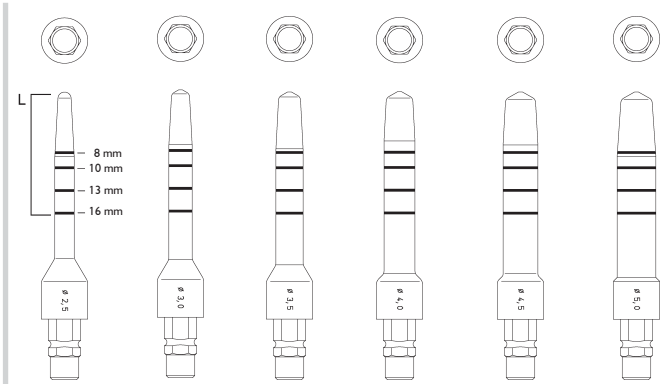

Einschlagwerkzeug
Impact tool
Herramienta de impacto

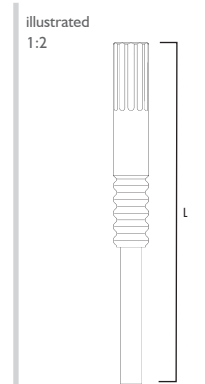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- Optimal system for the easy execution of an internal sinus lift
- Controlled minimal invasive procedure
- Vertical and horizontal bone condensing
- Optimal utilization of existing bone substance
- Suitable for preparing the bone for subsequent insertion of all standard forms of intraoral dental implant fixtures

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Indirect Sinuslift System

85FL071-0115

Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

CE 0044

Fig.	SL25	SL30	SL35	SL40	SL45	SL50	SL0
Size ²	025	030	035	040	045	050	
Length mm	16.0	16.0	16.0	16.0	16.0	16.0	100.0

²Largest working part diameter in 1/10 mm

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The Bone Management® system Internal-Lift-Control is a system for performing internal sinus lifts where the bone in the implant site is shifted upwards in direction of the maxillary sinus. Summers has developed this technique with transcrestal access to the maxillary sinus. Augmentations of smaller vertical defects (3 mm-4 mm) in the sinus at a residual bone height of more than 6 mm are deemed an indication for this procedure with an implantation at the same time depending on the case situation.

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The use of a pre-punch creates a guide channel to prevent any slipping of the twist drill.

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Pre-drilling is then performed with a suitable pilot drill with depth markings. The depth and diameter depend on the available vertical and horizontal bone dimensions, whereby the drill hole should correspond to up to two thirds of vertical bone height (check with x-ray). In as far as no simultaneous horizontal condensation is required, use the expanders to widen the drill hole to the implant diameter according to the manufacturer's protocol.

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A further successive preparation of the implant site shall take place with increasingly larger elevators. Treatment takes place up to about 1 mm beneath the sinus floor. The sinus floor including the apically condensed bone is driven cranially

into the sinus with the aid of the previously used elevator. The Schneiderian membrane is thereby raised away from the maxillary sinus floor.

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During the single-stage procedure, the newly established space is filled with a graft and an immediate insertion of the implant is performed.

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During the two-stage procedure, as an initial step, the newly established space is filled with a graft. The implantation takes place as part of a second intervention as soon as the new formation of bone has come to an end.

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- Residual bone height under 5 mm
- Not recommended in the case of complex morphology of the sinus floor (septum)
- Dentoalveolar growth which has not come to an end (exception: Cases in which no dentoalveolar growth can be expected, e.g. ectodermal dysplasia)
- Active infections as well as pathological processes

CAUTION: Lifting the Schneiderian membrane must take place very carefully with a low level of pressure. A perforation should be avoided.

NOTE: Related contraindications and further instructions that are deemed set-specific and generally valid can be found at the download area of our website at "Application and Safety Instructions for Bone Management® Systems".

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All products delivered are unsterile, therefore, before initial and each further potential application, products should be treated (cleaning/disinfection/sterilization).

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