

# starMed PROtect-shield

Face protection shield for glasses and magnifying systems

Please make sure that the front side of the shield is protected from scratches. Always put the shield face up like shown in picture (7). **Not like in picture (8).**

The round edge of the shield is ABOVE picture (1).

Mount the shield on the left and right of your temple with the pre-assembled clamp adapter's picture (3 & 4). Start with cable duct side (if available, usually right side). The cable duct should be positioned in the recess of the clamping shell picture (6 & 10). To do this, disassemble the clamp adapter picture (3 & 3a). Loosen the knurled screw until the temple can be threaded picture (5).

When both sides are attached to the temples, pay attention to a symmetrical positioning of the clamping shells picture (10). Before fixing, check that there is enough space for magnification glasses with light when the shield is folded down. If not, you can place the magnification glasses closer to the face or to the shield using the 3 holes in the shield change the axis of rotation picture (9).

Correct axes set - then fix the shield with the two thumbscrews. Now you can use the magnifying glasses with the PROtect-shield and fold it up and down picture (1 & 2).

**! Always use a face mask with the shield!**

Follow the cleaning and disinfection instructions on the back page

THANKS PAT.pending

Support call (303) 268-5400


MEISINGER USA

10150 E Easter Ave

Centennial, CO 80112

[customerservice@meisingerusa.com](mailto:customerservice@meisingerusa.com)

[www.meisingerusa.com](http://www.meisingerusa.com)

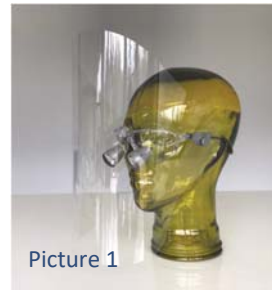


Picture 1



Picture 2



Picture 3

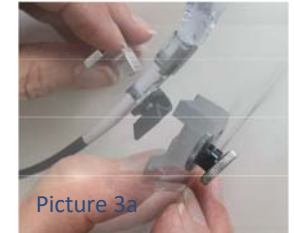


Picture 3a



Picture 4



Picture 5

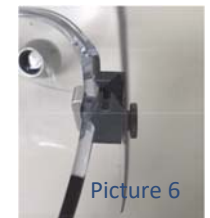


Picture 6




Picture 7



Picture 8

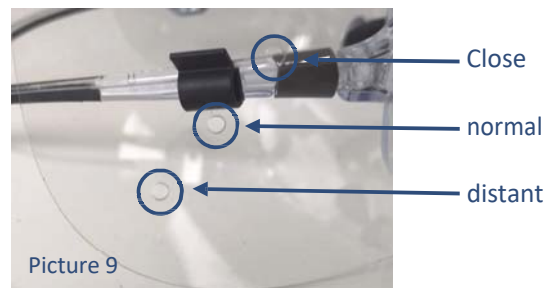

Picture 9

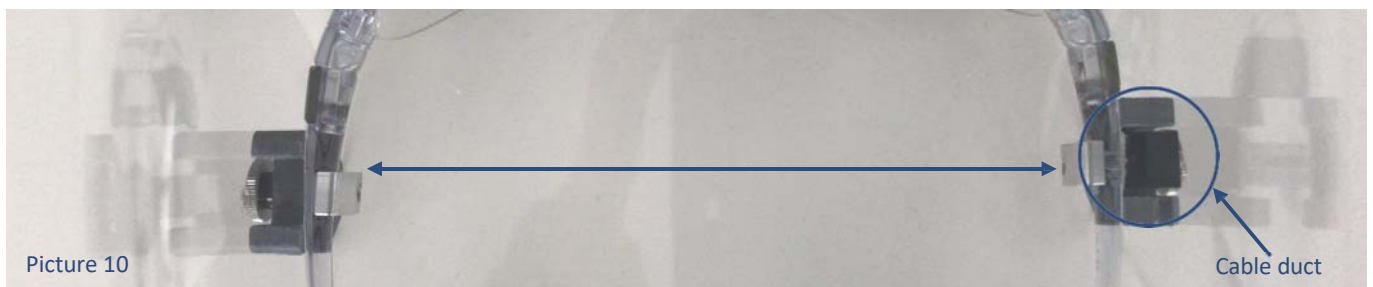

Picture 10

Cable duct



star**Med PRO**tect-shield cleaning and disinfection instructions:

**PRO**tect-shield from star**Med** has a pore-free surface that dirt can hardly hold. Dust or dirt can be removed with running water, a soft cloth or sponge.

**NEVER rub or wipe dry (risk of scratching).**

### **Cleaning**

- Remove the shield from the magnifying glasses
- Rinse the shield under running water without cleaning agents
- Apply dishwashing liquid to the shield by hand or with a soft cloth
- Allow persistent dirt to soak (**do not scratch**)
- Rinse off detergent under running water / rinse off
- Dry the shield with a soft cloth (**no paper**)
- If necessary, disinfection can be performed (according to the instruction)

### **Disinfection** with non-alcoholic disinfection agent / wipes

- First clean the shield as described above
- Spray on alcohol-free disinfectant or wet the shield completely with non-alcoholic disinfectant wipes
- Observe the exposure time according to the manufacturer
- Rinse the shield and at the same time the disinfectant
- Dry the shield with microfiber cloth or blow off (**no paper**)