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- The Scalino kit allows for a simple and controlled axial wall reduction for a modified shoulder style crown preparation
- Supported, secure depth marking (cutting depth 4.0 mm, 6.0 mm, 8.0 mm)
- Minimal loss of substance due to torpedo-shaped diamond
- Excellent use also during the filling stage due to the slight raising of the occlusal surfaces
- A simple and controlled procedure providing predictable results

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#### General instructions:

- All products delivered are unsterile therefore, before initial and each further potential application to be treated (cleaning / disinfection / sterilisation).

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The SCALINO set by Meisinger is specifically designed to be used for the controlled and supported shoulder preparation of tooth stumps. In particular, it has been developed in view of the increasing popularity of all-ceramic restorations. The special features of this system are the SCALINO diamonds, which, due to their safe support, easy guidance and defined depth achieve a perfectly prepared stump or crown construction. The set includes SCALINO diamonds of three different lengths (4.0 mm, 6.0 mm and 8.0 mm) with two different granulations (medium and coarse). In addition, the set contains diamonds to prepare the occlusal surfaces in the lateral tooth area as well as the palatal and lingual surfaces in the frontal tooth area. A separator for difficult interdental preparations completes the set.

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The aim is to prepare the axial walls to allow for the fabrications and placement of a modified should style crown recommended for all ceramic restorations.

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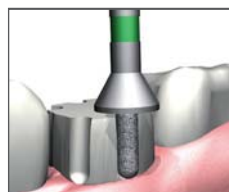
As a first step, a diamond (i.e. 830) is used to reduce the occlusal surface, which, at a later stage, will serve as support for the shoulder of the SCALINO diamond.

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
The choice of the correct length of the SCALINO diamond depends on the final length of the crown. In order to achieve an even shoulder height, it is recommended to initially pre-grind some depth markings, which will subsequently be combined with each other.

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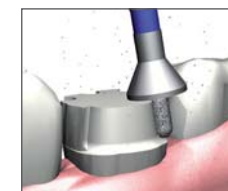Now, the prepared tooth stump can simply be circumnavigated with the instrument while the shoulder of the diamond is supported by reduced occlusal surface of the tooth.

Picture 1: Shoulder by SCALINO Diamond - 8 mm  
Picture 2: Shoulder by SCALINO Diamond - 4 mm

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